



IntraGen

Experience !  
Smart **Grid RF™** !!

**Grid RF Technology**  
Grid fractional Radio Frequency™

Real time Impedance  
Feedback System

IntraGen  
Pulse Mode

IntraGen

#### Technical Specification

Applied Frequency	IntraGen : 6Mhz
RF Output Mode	Monopolar
RF Energy per Level	User Mode : 5Watt ~ 200Watt IntraGen Mode : T Level ~ 7Level
User Mode	ON TIME : 0.1s ~ 2.0s (0.01s unit) OFF TIME : 0.1s ~ 2.0s (0.01s unit) Number Of Pulses : 1 ~ 4 Watt : 5W ~ 200W (1W Unit)
Small Tip	7mm X 7mm (0.32cm²)
Full Tip	15mm X 15mm (1.69cm²)
Output Switch	Handpiece Switch or Foot Switch

#### System Specification

Weight	28Kg
Dimensions	370mm x 450mm x 810mm (including stand 1,240mm)
Handpiece Hose Length	2.3M
Power Consumption	220V, 600VA (For Domestic Sales) 90V ~ 240V (Free Voltage), 600VA (For Overseas Sales)

**Skinnovation**  
*We Breathe Skin*



Skinnovation Pvt. Ltd.

502 / 503, Shreya House, Pereira Hill Road, Off Andheri - Kurla Road, Opp Gurunanak Petrol Pump,  
Andheri (E), Mumbai 400 093 | PH : 28246384, 28386327, Mob : +91 9769913008 | [www.skinnovation.in](http://www.skinnovation.in)



IntraGen

**Jeisys**





## Range of Treatment

The treatment by the IntraGen Grid RF Technology is proceeded through 3 passes. It can make better efficacy.

1 PASS 2 PASS 3 PASS

# IntraGen


## IntraGen Treatment Tip Set

IntraGen provides various treatment from a very shallow or very small treatment area to a very large or deep treatment area. Each IntraGen Tip is sterilized.

IntraGen tip set is equipped with RF Card and tip. Each set is composed optimized parameter for high performance.

## Grid RF™ Technology

Grid fractional Radio Frequency™

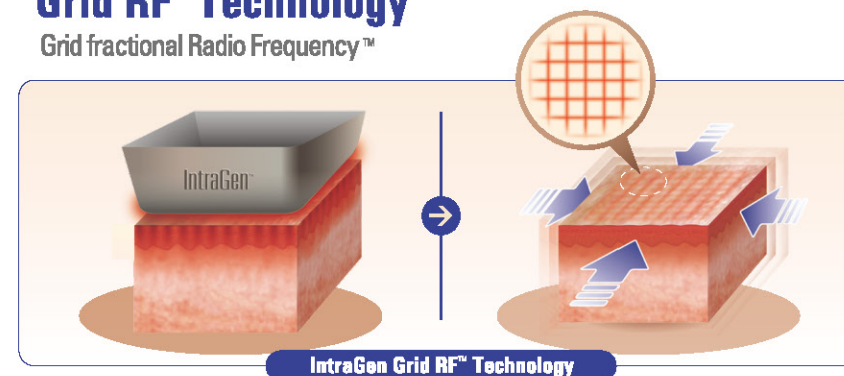

The IntraGen system includes a 6MHz generator that delivers RF energy through a sophisticated treatment tip. The IntraGen tip at the end of the IntraGen handpiece treats on selected epidermal area. Grid

fractional radio frequency energy creates thermal grid shapes on epidermis and thermal uniform zone in upper dermis during RF emission.

## IntraGen Pulse Mode




Effective use of Grid RF™




## Real time Impedance Feedback System

Consistent energy emission to heat up dermal area

Constant and reliable treatment in any impedance value

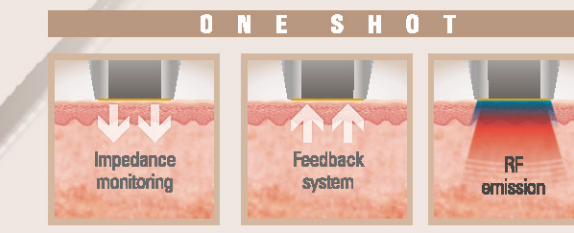

## IntraGen Tip Size

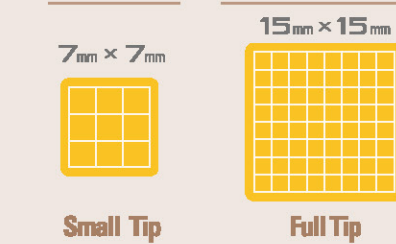

“  
More patients are satisfied by  
Grid RF technology of IntraGen  
”

Experience !  
Smart Grid RF™ !